

GENERAL

The following product care recommendations are outlined for maintaining a safe, hygienic, and comfortable work environment when using Texray products MindPeace and HeadPeace.

REGULAR INSPECTION

Products should be inspected thoroughly upon receipt and at regular intervals, following your hospital protocol. When the products are first received and before they are put into service, they should be tested for integrity by fluoroscopy or any other suitable methods, following hospital protocol or by advice from your radiation safety officer. Means for attachment such as Velcro or magnet bands shall also be inspected thus to assure safe closure.

STORAGE

When storing, ensure that the Velcro closure is properly attached, this to ensure that the Velcro do not risk damaging the fabric or binding.

PROPER CARE AND HANDLING

Care should be taken to avoid sharp objects and edges as they may cause unseen damage to the protective material. Avoid exposure to extreme temperatures. If the product is exposed to extremely hot or cold environment, it should be acclimatized to ambient temperature before use.

GENERAL CLEANING INSTRUCTIONS

Before cleaning and disinfection of the products, all soil and biofilm should be removed. This should be done by using a damp, lint free cloth or a soft brush. Cold water has shown to be effective in removing proteins.

- Never immerse or soak the product with fluid.
- Never use autoclave, machine laundering or dry cleaning.
- Never use alcohol or bleach containing liquids.

CLEANING

Use a damp cloth with lukewarm water and a mild environmentally friendly (non-ionic/anionic) detergent to clean the product. Do not rub extensively on single spots. Let the product air dry in ambient temperature.

DISINFECTION

If disinfection is needed, use a non-abrasive, non-alcoholic, non-bleach damp wipe. Scrub small area of the product using a circular overlapping motion. Avoid applying pressure when using disinfectant wipes.

Any of the following disinfection fluids are recommended:

Microbac forte	Hartmann	Quaternary compound
Kohrsolin FF	Hartmann	Quaternary compound

Cleaning and disinfections tests have been carried out in-house and do not correspond to any certified or standardized test procedures, hence Texray cannot issue a release, but only a recommendation.

Since Texray has no influence on the production as well as the raw materials of the disinfectant, we must decline any guarantee on the resistance of our protective clothes to the disinfectant. The effectiveness of the surface disinfectant has not been considered. The instructions for use of the disinfectant must be adhered to. For these reasons, Texray excludes any warranty or liability for subsequent damages.

If you have any further questions about disinfectants or if your clinic or local regulations do not allow the disinfectants listed above, please do not hesitate to contact your sales representative or Texray at info@texray.io.